

In with ALLEVYN[®] Plus Cavity. Out with exudate.

With its unique bi-laminate sodium polyacrylate technology, ALLEVYN Plus Cavity maximizes absorption to maintain the ideal moisture level for faster healing of cavity and other wounds.

Increased exudate absorption means increased wear time and decreased maceration. Plus, the dressing conforms to the shape of cavity wounds and stays intact upon removal.

For heavily draining wounds, put in ALLEVYN Plus Cavity and pull out the exudate.



For cavity and other heavy exudate wounds, ALLEVYN[®] Plus Cavity provides excellent absorption

Maintains optimal moisture balance

Unique bi-laminate technology of embedded sodium polyacrylate absorption molecules.

For all heavy exudate wounds...

Quick, persistent absorption

Highly absorbent foam that holds high levels of exudate, even under compression.

No outer film layer combined with excellent vertical wicking capacity reduces pooling and maceration.

For cavity wounds...

Conforms to the shape of cavity wounds

Expands to the wound surface for protection against bridging and premature or incomplete closure.

Easy to remove from healing cavities

Stays intact upon removal from cavities, minimizing pain and disruption of healing tissue.



Respond to the wound healing environment with ALLEVYN.

Go to www.allevyn.com for more information, or call 1-800-876-1261 to request a free sample.

Product	Description	Pcs/Pkg	Pkgs/Unit	Order Unit
66047571	2"x 2 3/8"	10	10	CA
66047573	4"x 4"	5	20	CA
66047574	6"x 8"	3	16	CA

Indications

ALLEVYN Plus Cavity is a dressing for the management of deep secondary healing wounds, particularly for the following indications: stage III and IV pressure ulcers, deep leg ulcers, cavity wounds, excisions, and post-op wound dehiscence.

Contraindications

Do not use in deep cavity wounds with small openings.

Application

Prepare and clean the skin surrounding the wound area. SKIN-PREP[®] wipes may be used if appropriate. Select appropriate dressing size to fill approximately 50% of cavity. Product may be cut to accommodate size requirement. Fold product onto itself and gently insert into cavity area. Upon release, product will slowly conform to the wound cavity shape. Product may be retained using appropriate cover dressing (ALLEVYN, OPSITE[®]). For removal, gently remove the saturated product and cleanse area.

Precautions

Do not use any ALLEVYN products with oxidizing agents such as hypochlorite solutions or hydrogen peroxide solutions.

Wound Management

Smith & Nephew, Inc.
970 Lake Carillon Drive
Suite 110
St. Petersburg, FL 33716
USA

Customer Care Center
1 800-876-1261
T 727-392-1261
F 727-392-6914

www.smith-nephew.com
www.allevyn.com