

ALLEVYN[®] SACRUM

Hydrocellular Adhesive
Sacrum Dressing

ALLEVYN[®] Sacrum offers secure protection

In a hostile environment, waterproof and bacteria-proof ALLEVYN Sacrum offers stay-put performance that promotes a moist wound environment for rapid healing.

ALLEVYN Sacrum also provides cushioned hydrocellular comfort, and its tapered edges and secure adhesive guard against roll-up.

For sacral wounds, you can feel secure with ALLEVYN Sacrum.



For the extreme demands of sacral area wounds, ALLEVYN[◇] Sacrum is a powerful guardian

Stays put in the high-friction sacral area

Long wear time— anatomical shape and secure adhesive help the dressing stay in place.

Tapered edges protect against roll-up and accidental removal.

Waterproof and bacteria-proof to protect the wound.

Maintains optimal moisture balance


Unique tri-laminate polyurethane hydrocellular foam promotes a moist wound environment for rapid healing.

Breathable outer film layer allows fluid to evaporate from dressing.

Designed for a perfect fit

Shaped for a snug fit to protect against urine and other wound contaminants.

Thick foam pad provides a cushion of comfort.

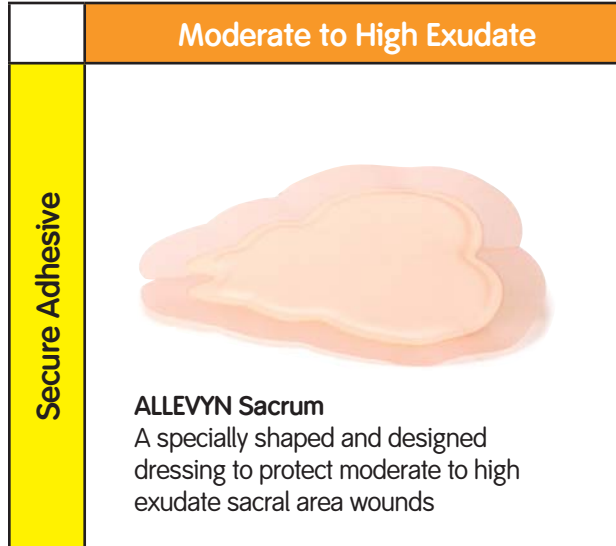

Respond to the wound healing environment with ALLEVYN.

Go to www.allevyn.com for more information, or call 1-800-876-1261 to request a free sample.

Product	Description	Pcs/Pkg	Pkgs/Unit	Order Unit
66000700	6 3/4" x 6 3/4"	10	8	CA
66000451	9" x 9"	10	8	CA

When positioning ALLEVYN Sacrum dressings, place the narrow end of the dressing a minimum of 2 cm above the anal sphincter, then smooth the dressing over the sacral area.

Precautions

Do not use ALLEVYN dressings with oxidizing agents such as hypochlorite solutions or hydrogen peroxide, as these can break down the absorbent hydrocellular component of the dressing.

In common with all adhesive products, some cases of irritation and/or maceration of the skin surrounding the wound have been reported. Infrequently, cases of sensitivity to the dressing have been reported. It should be noted that inappropriate use or too frequent dressing changes, particularly in patients with fragile skin, can result in skin stripping.

If reddening or sensitization occur, discontinue use and consult a healthcare professional.

Indications

ALLEVYN Sacrum may be used in the management of exuding, granulating wounds on the sacrum, healing by secondary intent. It can be used in conjunction with INTRASITE[®] Gel for necrotic or sloughy wounds.

Application

Prepare and clean the skin around the wound. SKIN-PREP[®] wipes may be used if appropriate. Peel back the protector papers and anchor the dressing over the wound. Smooth the dressing into place, ensuring that the edges of the dressing are not wrinkled.

Wound Management

Smith & Nephew, Inc.
970 Lake Carillon Drive
Suite 110
St. Petersburg, FL 33716
USA

Customer Care Center
1 800-876-1261
T 727-392-1261
F 727-392-6914

www.smith-nephew.com
www.allevyn.com