

The beach chair positioner that sits well with patients, surgeons, anesthesiologists and OR staff.






Smith & Nephew's T-MAX Beach Chair Positioner places the surgeon, anesthesiologist and surgical staff in a good situation for beach chair shoulder procedures, providing the tools they need to easily position patients. T-MAX offers a simple and intuitive operation. It's adjustable



Hydraulic Assist Mechanism, 0°



Hydraulic Assist Mechanism, 45°

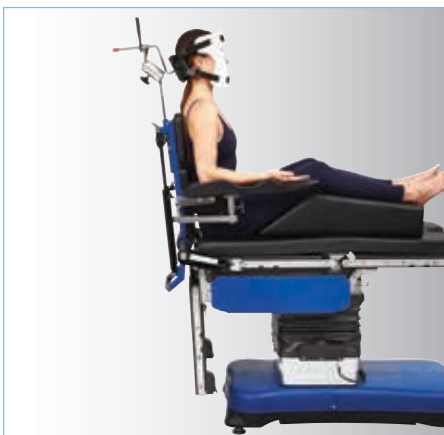

Hydraulic Assist Mechanism, 90°

**Unique, double ball-and-socket design** offers infinite positions of tilt and rotation, maintaining the patient's head and airway in a neutral position throughout the procedure. The disposable face mask allows secure attachment to head rest. Alternately a optional reusable Gel Face mask may be used.


**Adjustable lateral support** secures patient within the beach chair and provides countertraction when needed. Adjustable height and width accommodate a variety of patient sizes.

**Hydraulic assist mechanism** assists in raising and lowering patient to any angle between 0° and 90°.

**An unobstructed posterior view** is provided for the surgeon because the T-MAX is placed at the end of the OR table. This also allows the anesthesiologist easy access for monitoring the patient.



legs allow the T-MAX to attach to many commonly used OR tables and securely positions patients of up to 350 lbs., with a minimum height of 5'2" (1.57 m)<sup>1</sup>. Securely holds the patient while positioning and re-positioning between 0 and 90 degrees. Built-in lateral supports provide sturdy counter traction when distracting the operative arm.

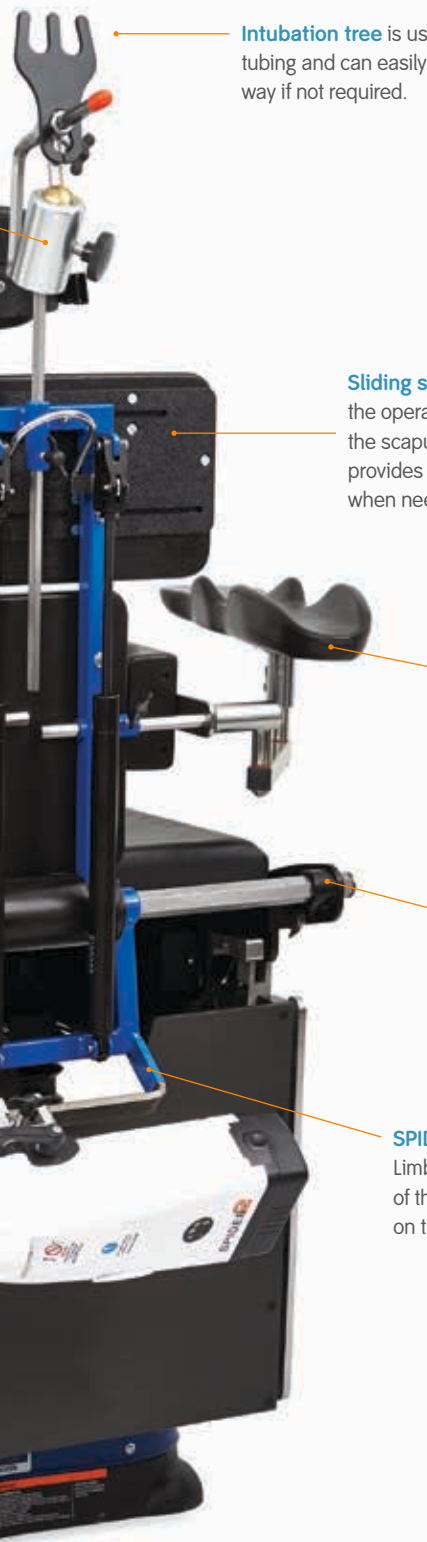

**Intubation tree** is used to hold intubation tubing and can easily be rotated out of the way if not required.

**Sliding shoulder pad** offers full exposure of the operative shoulder to the medial border of the scapula in arthroscopic procedures. It also provides full support of the posterior shoulder when needed during arthroplasty surgery.


**Non-operative arm board** provides support and adjustability for the non-operative arm, eliminating the need to attach an arm board elsewhere to the OR table.

**Lateral adjustment** allows the T-MAX to slide laterally on the frame to either side of the OR table, to position patients closer to the surgeon.


**SPIDER rail** allows SPIDER and SPIDER2 Limb Positioners to be secured to the base of the chair – a unique feature found only on the T-MAX Beach Chair.




Head Rest, extended

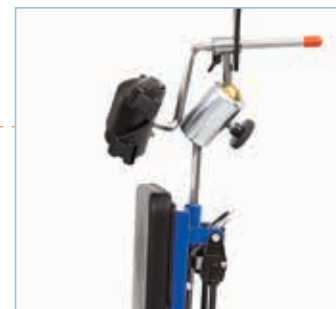

Head Rest, retracted

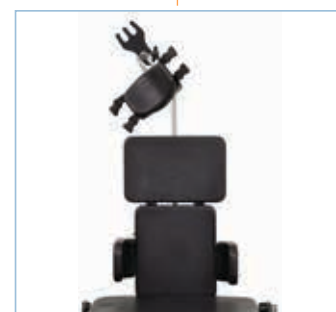

Head Rest angled for left shoulder access

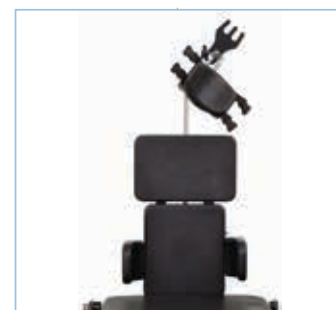

Head Rest angled for right shoulder access



Sliding Shoulder pad positioned to provide support for shoulder arthroplasty



Sliding Shoulder pad positioned to provide full exposure to the medial border of the scapula for shoulder arthroscopy

1. Data on file: Table test calculations document updated 8/19/2014.



For more information, contact your sales representative  
or visit [smith-nephew.com](http://smith-nephew.com).

## Ordering Information

### T-MAX Beach Chair Positioner

Reference #	Description
7210551	T-MAX Beach Chair Positioner Includes: Chair and Pad Assembly Head Positioning Assembly Table Leg Assembly Knee Bolster Lateral support (2 Units)

### Disposable Accessories

7210559	Disposable Face Masks (case of 8)
---------	-----------------------------------

### T-MAX Table Rail Clamps and Leg Kits

7210554	North American Rail Clamps (2 Units)
7211099	European Square Rails Clamps (2 Units)
7211096	JE Adaptor Leg Kit
7211097	JE Rail Clamps (2 Units)
7211098	DE Adapter Leg Kit
7211100	Denyer Rail Clamps (2 Units)

### T-MAX Accessories

7210552	Adjustable Arm Board
7210553	T-MAX Cart
7210560	Knee Bolster
7210563	Drape Bar
7210564	Lateral Support Knobs (2 Units)
7210555	Intubation Pad
7210556	Gel Face Mask
7210557	Extended Lateral Support (2 Units)
7210558	Mask Straps (pair)
7210568	Table Stabilization Straps
72203415	Non-Operative Arm Board
72203318	Adjustable Lateral Support

## Highlight T-MAX Accessories

### 7210560 – Knee Bolster

The knee bolster helps prevent the patient from sliding down the OR table.



### 7210563 – Drape Bar



### 7210555 – Intubation Pad

The intubation pad (optional) provides anesthesiologists with a broad, flat surface for ease-of-use with intubation.



### 7210556 – Gel Face Mask



### 72203415 – Non-operative Arm Board



### 72203318 – Adjustable Lateral Support

