

# Quick Instruction Guide

Read prior to use.



Remove EXOGEN device, strap and gel bottle from black tote.

The following is a list of symbols you may see and instructions on what to do if they appear:

-  Attention, consult **accompanying documents**.
-  A blinking addition (+) symbol with an audio tone means you need to add gel to the transducer face.
-  Pulsing Treating Symbol Indicates device is operating properly.
-  Treatment Stop or Treatment Lockout symbol.

For Technical Support Call: USA 1-800-836-4080, Europe (+49)-746-22080, In Other Countries USA (+1)-901-396-2121

Orthopaedics Trauma & Clinical Therapies - Smith & Nephew, Inc., 1450 Brooks Road, Memphis, TN 38116, USA  
 Telephone: 1-901-396-2121, Information: 1-800-821-5700, Orders/inquiries: 1-800-238-7538  
 www.smith-nephew.com

# Device operating instructions

## Assembling and placing strap



1. Hold strap with cap side up and pull end of strap through bottom of loop.



2. Squeeze sides of cap to open.



3. Position the strap with the cap placed directly over fracture site as marked by your physician.



4. Tighten strap securely utilizing hook and loop.

## Application of gel to transducer


5. Remove top from gel bottle



6. Hold transducer where cable meets the transducer head. Place finger on pump and press down to apply one pump of gel onto transducer face.



7. Place transducer into the RAF, gel should be touching skin.

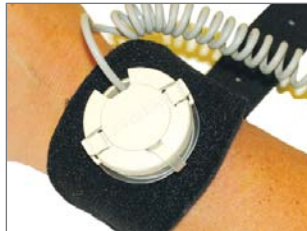

8. Align U shape in top of cap with transducer cord and snap cap shut.

9. Replace top on gel bottle.

## Conducting your treatments



10. Press On/Off button once to begin treatment.



11. EXOGEN 4000+<sup>®</sup> device will conduct a brief self test, run for 20 minutes, and then turn itself off.

## After treatment has completed

12. Open cap



13. Remove transducer from RAF and clean any excess gel from transducer head with a soft cloth.



14. Remove strap from treatment site. Clean excess gel from your skin and strap with a soft cloth.

15. Place unit and accessories into black tote for storage until next treatment time as prescribed by your doctor.